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The ISOQUAD IQ200 and IQ810 Bed Isolation Modules are flexible, low cost, and easy ways to isolate potentially infectious patients right on their bed or gurney, and protect other patients and staff from exposure to infectious airborne pathogens.

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Fast Setup:	Pop up in seconds. No tools required.
Economical:	The low-cost patient isolation solution.
Secure:	"Dirty-to-clean" directional airflow effectively contains pathogens.
Versatile:	Fit virtually any size bed or gurney.
Easy to Store:	Compact, soft carrying case with handle included.
User Friendly:	Sized right for easy staff and equipment patient access.

ISOQUAD® Bed Isolation Modules

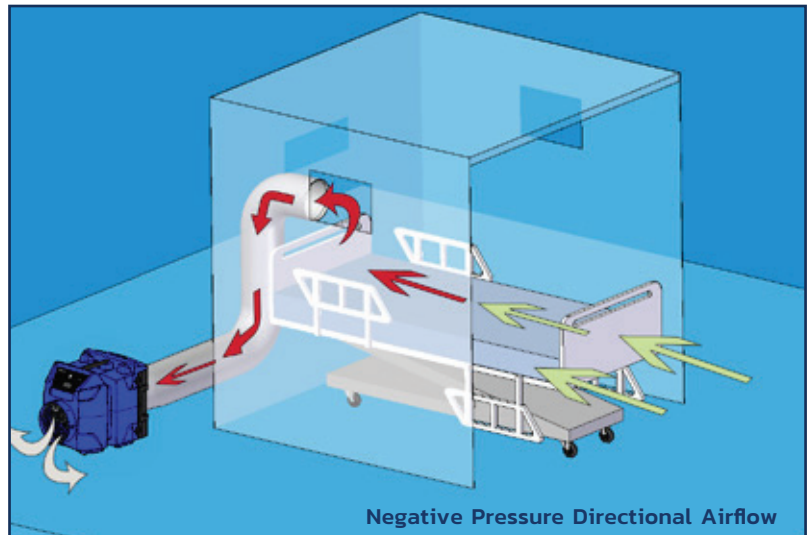
Fast, Cost Effective, In-Place Patient Isolation

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Negative Pressure Infectious Patient Isolation

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To use the ISOQUAD IQ200 or IQ810 module in the negative pressure mode simply remove it from its convenient carrying pouch and pop out the side of the module to make it operational. For the IQ200, connect flex duct between the PVC collar at the rear of the module and the inlet collar of the Portable Air Scrubber (PAS) / For the IQ810, zip the front flap closed and connect flex duct to ensure that “dirty” air pulled from the ISOQUAD module is HEPA filtered. This creates negative (lower) air pressure within the enclosure so that infectious droplets expelled by the patient are captured by the Portable Air Scrubber and only HEPA-clean air is exhausted back into the room.

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
Positive Pressure Protective Isolation

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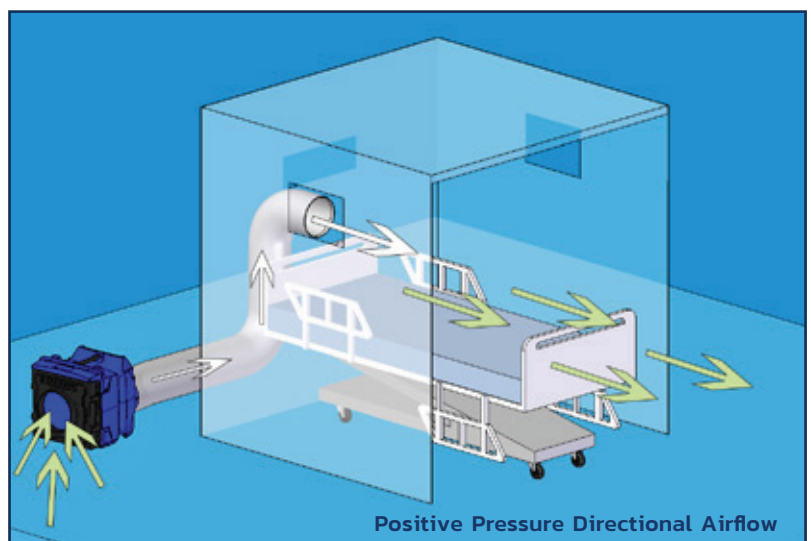The versatile ISOQUAD Module also effectively creates positive pressure Protective Environments (PE) for sensitive or immune-compromised patients. In this configuration flex duct is connected between the PVC tube at the rear of the enclosure and the exhaust collar of the Portable Air Scrubber, so the HEPA-filtered exhaust air blows directly into the enclosure, creating positive pressure and head-to-foot directional airflow.

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ISOQUAD® IQ200 Bed Isolation Module
(with open front)

ISOQUAD® IQ810 Bed Isolation Module
(with front flap rolled up)

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PREDATOR® Portable Air Scrubbers (Molded High-Density Polyethylene Cabinets)

Made with ultra-tough and lightweight high-density polyethylene cabinets, PREDATOR® rotational-molded portable air scrubbers set new industry standards for power, efficiency, mobility, and durability.

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The compact PREDATOR®750TF can be easily hand carried to the job site, or wheeled in on a dolly. At peak airflow, it can maintain 6 ACH in areas up to 6,000 – 7,500 ft³.

- Variable speed control, 200 – 750 cfm
- High-capacity mini-pleat HEPA w/injection-molded frame
- Convenient, built-in carrying handle
- Dimensions (L x W x H) – 25.5" x 19.5" x 20.5"
- Portable and incredibly lightweight (35 pounds with filters)
- Easy access for fast pre-filter and HEPA filter replacement

FILTERS:

- Stage 1 pre-filter – 1" coarse particulate pre-filter
- Stage 2 pre-filter – 2" pleated particulate pre-filter
- Optional Stage 2 Vapor-Lock® carbon filter
- Final stage True HEPA filter – Tested and certified to an efficiency of 99.97% or higher against 0.3 micron size particles

Ordering Information

IQ810-PD7

ISOQUAD Complete Patient Bed Isolation Kit, Includes: 1 ea. IQ810 ISOQUAD Module, 1 ea. PRED750TF PREDATOR Portable Air Scrubber, 1 ea. Flex Duct, & 2 ea. Flex Duct Attachment Clamps.

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PREDATOR® Portable Air Scrubbers (Molded High-Density Polyethylene Cabinets)

Made with ultra-tough and lightweight high-density polyethylene cabinets, **PREDATOR®** rotational-molded portable air scrubbers set new industry standards for power, efficiency, mobility, and durability.

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The Predator1200UVTF provides users with the ultimate in technology, performance, mobility, and versatility.

- Variable speed settings, 300 – 900 cfm
- Air changes per hour – 6 ACH @ 6,000 – 8,000 ft³
- Dimensions (L x W x H) – 22" x 24" x 42"
- Net weight with filters – 70 pounds
- Built-in dolly – Two-wheel dolly consisting of 8" solid, non-marking rubber rear wheels, 360° swivel front casters and a secure top handle for gripping
- Simple filter access – Convenient hinged rear pre-filter access door provides quick access to filters
- Easy-to-use controls
- Internally mounted UVGI lamp to kill bacteria

FILTERS:

- Stage 1 pre-filter – 1" coarse particulate pre-filter
- Stage 2 pre-filter – UV-resistant 2" pleated particulate pre-filter
- Stage 3 high-output UVGI lamp – An internally mounted 39W UVGI lamp located below the pre-filters
- Final stage True HEPA filter – Tested and certified to an efficiency of 99.97% or higher against 0.3 micron size particles

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HEPA-CARE® SYSTEMS FOR PATIENT ISOLATION

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Abatement Technologies offers portable in-room HEPA filtration systems for health care facilities.

- State-of-the-art touch screen/pad controls
- Individually tested and certified HEPA filters
- Certification as Class II Medical Devices under USFDA 510K

These powerful and feature-rich models can be configured for negative pressure or recirculation air purification.

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The lightweight HC400FTF and HC400FUVTF units roll easily within a healthcare facility to ■ convert standard patient rooms into CDC-compliant, negative pressure patient isolation rooms ■ provide recirculation air purification ■ increase infectious patient surge capacity in case of emergency.

- Adjustable 4-speed fan, 200 – 750 m³/hr (125 – 450 cfm)
- Peak airflow of 750 m³/hr can provide 12 ACH in designated areas
- Quiet : 37 dBA to 53 dBA @ 1.5 m (5 ft) with baffle attachment
- HEPA filter individually tested & certified to IEST-RP-CC007 and up to 99.97% removal at 0.3 microns
- HC400FUVTF model adds an extra level of protection with an integrated UV lamp inside the standard HEPA filter

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The compact HC600FTF and HC600FUVTF units are economical options to the popular HC800FTF model.

- Five airflow settings from 150 cfm to 675 cfm
- When used in the negative pressure mode, the units can generally provide 12 ACH or more in rooms as large as 2,500 – 3,000 ft³
- Net weight with filters : 63 pounds
- Quiet : 52 dBA to 58 dBA @ 5 ft
- Dual exhaust outlets (top and rear)
- 99.99% HEPA filter with closed-cell neoprene gasket seal
- HC600FUVTF model has a powerful 39W UVGI lamp built-in to the HEPA filter

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รุ่น HC800FTF

The powerful HC800FTF is the 'gold standard' for portable, in-room systems for patient isolation. It offers unmatched features and performance.

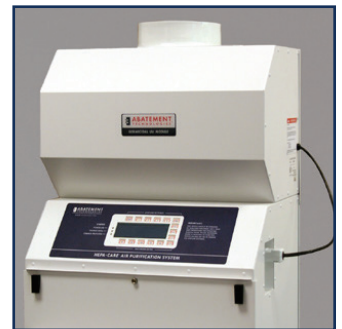
- Five airflow settings from 300 cfm to 900 cfm
- 12+ air changes per hour in rooms as large as 3,500 - 4,000 ft³
- When not in use for negative pressure patient isolation, the HC800FTF can function as a recirculating HEPA air cleaner, reducing airborne pathogen counts in high risk areas such as ER waiting rooms and treatment areas
- Quiet : 52 dBA to 59 dBA @ 5 ft
- Pharmaceutical-grade HEPA filter with silicone-gel seal and anodized aluminum frame
- Portability - Narrow, upright design with two smooth-rolling fixed casters and two swivel casters with foot brakes make it easy to deploy

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The UV800F is designed specifically for use with the HC800FTF, mounting onto the top exhaust outlet. Air flowing through the UV800F is irradiated with a high dosage of germicidal UV (UV-C) energy.



- Plugs directly into a receptacle on the HC800FTF
- On/off switch so HC800FTF can be run with or without UV-C
- Safety interlock switch protects against UV exposure

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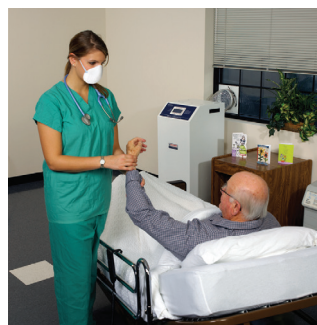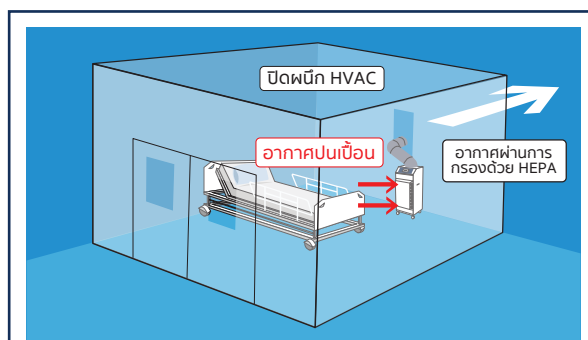NEGATIVE PRESSURE ISOLATED MODE WITH HEPA-FILTERED AIR EXHAUSTED DIRECTLY OUTDOORS (SEALED HVAC)
In this configuration, the unit pulls in contaminated air, filters out contaminants and propels the filtered air outside the environment to negatively pressurize the room.

Note : Use airflow ratings and air changes per hour (ACH) as a guideline only. Estimates are based on factory and independent testing @ 120 VAC with an air straightener and a traverse of readings taken with a computing vane-anemometer. Actual results may vary.

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LEAFPOWER
SAVE THE WORLD • SAVE YOUR LIFE

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